



Factor II, Incorporated

*****DO NOT USE THIS PRODUCT WITH SOFT CONTACT LENSES*****

FOR SCLERAL CONTACT LENS REMOVAL - Squeeze the tapered handle of the DMV Scleral Cup and carefully apply it to the surface of the scleral contact lens. When the suction cup has been seated properly on the lens, release the squeeze pressure to apply and gently remove the contact lens from the eye. You may wish to place the Scleral cup in the center of the lens. Remove the contact lens from the Scleral Cup by squeezing the handle again.

Precaution: In the event of mistaken application of the Scleral Cup to the eye, merely squeeze the tapered handle to release suction. Should the lens resist removal from the eye, release the Scleral Cup by squeezing the handle to reverse the suction. Consult your eye care practitioner for an alternate removal process.

FOR LENS INSERTION – Lightly squeeze the handle of the Scleral Cup and place the properly wetted contact lens on the open end of the cup. Gently position the contact lens on the surface of the eye. Squeeze the handle again to release the Scleral Cup from the lens.

Do not use this product with diseased, injured or otherwise compromised eyes.

FOR PROSTHETIC EYE REMOVAL – Squeeze the tapered handle of the DMV Scleral Cup and carefully apply it to the surface of the prosthetic eye. When the Scleral Cup has been seated properly on the prosthesis, release the squeeze pressure to apply and gently remove the prosthesis from the eye socket. Remove the prosthesis from the Scleral Cup by squeezing the handle again.

CLEANING – Wash the Scleral Cup frequently in warm water and allow to air dry. Do not use this product with soft contact lenses.

CAUTION – This product contains natural rubber latex which may cause allergic reactions. Independent laboratory testing finds no antigenic proteins in this material. Data on file at DMV corporate offices.

CHOCKING HAZARD: Keep this product away from small children.

See your eye practitioner at appropriate intervals.